


# Understanding Your **CORGI WHEELCHAIR**



The Walkin' Wheels **Corgi Wheelchair** is specifically designed to fit the Corgi. Featuring a wheelchair frame that is 3.75" longer than our standard small Walkin' Wheels, the corgi wheelchair is perfectly designed to fit the long torsoed breed and help your dog to maintain a sense of independence while strengthening their muscles.

**The rear legs of the Corgi Wheelchair have been Counterbalanced right from the manufacturer.**

What does this mean?

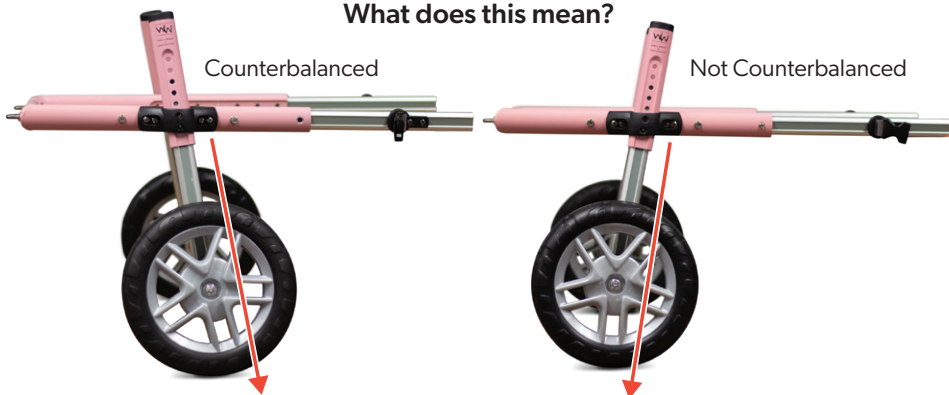

By angling the rear wheels slightly forward the wheelchair is better able to provide support to the midsection of your pup and support the natural balance of the body. This adjustment helps Corgi's walk easily in the wheelchair and prevents any dipping of the back which was a common fitting issue with the standard small frame for corgis.

**Walkin' Pets**  
by [handicappedpets.com](http://handicappedpets.com)

105 Route 101A, Suite 18  
Amherst, NH 03031  
[www.WalkinPets.com](http://www.WalkinPets.com)  
(888) 253-0777

**THE PROVEN LEADER IN PET MOBILITY**


## The Corgi Wheelchair comes with:

- 1 Medium Belly Belt
- Corgi Front Harness
- 7" Width Connector (installed)
- 10" Width Connector



## Adjusting the Front Harness:

Take the 3 black comfort sleeves off the harness so you can make sizing adjustments.

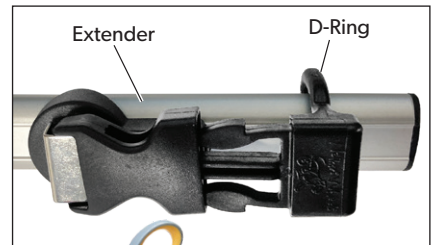

*Comfort sleeves can be cut for smaller pets and excess straps tucked in. This material will not fray.*

- Pet's head goes between the black strap and the blue strap.
- Blue strap goes over your pet's shoulders.
- Red strap goes behind the front legs and clips into the side of harness.
- Black strap goes under neck across front of chest.
- Side neoprene panels should be positioned over the shoulder blades.
- Male prong points to the back and clips in.
- All three straps should be snug.



105 Route 101A, Suite 18  
Amherst, NH 03031  
[www.WalkinPets.com](http://www.WalkinPets.com)  
(888) 253-0777

**THE PROVEN LEADER IN PET MOBILITY**